ostomygel

Gel para Ostomia
Gel para Ostomía/Ostomy Gel

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• English - User Manual

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English

FUNCTION AND INDICATION

OstomyGel solidifies the liquid present in the ostomized patient's pouch and eliminates the odor caused by these liquids.

THE LIQUID IN THE POUCH DOES NOT INCREASE IN VOLUME WHEN IT COMES INTO CONTACT WITH OSTOMYGEL.

COMPOSITION

SODIUM POLYACRYLATE, CHARCOAL POWDER.

MECHANISM OF ACTION

Sodium polyacrylate has a high capacity of water absorption, which occurs by osmosis. The osmotic pressure causes sodium polyacrylate to absorb water in order to balance the sodium ions concentration inside and outside the polymer.

The mechanism for removing odors consists of their physical adsorption by the charcoal, i.e. the molecules that generate the odors are attracted by the activated charcoal porosity and retained there by van der Waals physical forces.

FORMS OF PRESENTATION


OstomyGel is available in the following formats:

Format 1:

- 1, 3, 6, 12 or 20 bottles with net wt. 30 g (1.06 oz.);
- 1 User Manual;
- 1 measuring spoon.

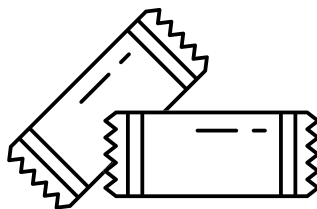Format 2:

- 10, 20, 25, 30, 50 or 60 sachets with net wt. 2 g (0.07 oz.);
- 1 User Manual.

STORAGE AND TRANSPORTATION

Keep the product protected from direct light and heat, and out of reach of children. Store at room temperature.

INSTRUCTIONS FOR USE

1. Wash and dry your hands carefully;
 2. Open the secondary packaging (box) and verify if the primary packaging (bottle) is intact;
 3. Bottle:
Open the primary packaging and add one or two doses (according to the size of the measuring spoon supplied) inside the pouch;
 - Sachet:
Add one or two sachets inside the pouch;
 4. Empty the pouch through its lower part, without applying pressure;
- The amount of material put into the pouch depends on its size and desired consistency.

GENERAL SAFETY INSTRUCTIONS

- Do not ingest the product;
- Keep out of reach of children;
- Keep in a dry place, at room temperature;
- Do not use after the expiration date.

SIDE EFFECTS

The are no side effects related to this product.

CONTRAINDICATIONS

The are no contraindications related to this product.

CAUTIONS

- Do not use after the expiration date;
- Keep in a dry place and away from light.

LIMITATIONS

The are no limitations related to this product.

SHELF LIFE

OstomyGel has a shelf life of 3 years from the date of manufacture. After opening the primary packaging (bottle), the product must be used within a maximum of 60 days.

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